

# Checking Your Blood Glucose Using TRUE2go™

*A guide for testing your blood glucose*

## Simple Steps to Test Your Blood Glucose

### Meter, Test Strips & System Components



### Blood Glucose Testing



Wash your hands with warm, soapy water.



Remove Test Strip from vial and close vial immediately. Insert Test Strip into Test Port with TRUEtest™ facing up. The Meter turns on.




Lance your finger.



Allow a blood drop to form, touch tip of strip to top of blood drop until dashes appear across Display.



In as little as 4 seconds, glucose result will be displayed.

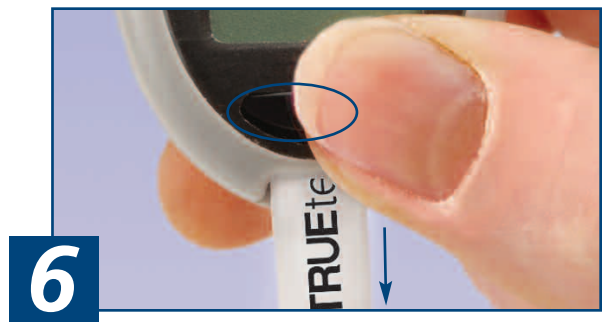

Hold Meter in a vertical position with Test Strip facing down. Press strip release button to discard the used Test Strip from the Meter.

This guide provides a brief overview on testing your blood glucose. For more detailed information on complete blood glucose testing procedure, consult "Testing Your Blood" section in the Owner's Booklet. For assistance using your blood glucose System, call our Customer Care Department at 1-800-803-6025.

# Control de glucosa en la sangre con TRUE2go™

## Medidor, tiras de prueba y componentes del sistema

*Guía para su prueba de glucosa en la sangre*

### Pasos simples para su prueba de glucosa en la sangre



### Pruebas de glucosa en la sangre



**1**

Lávese las manos con agua tibia y jabonosa.



**2**

Retire la tira de prueba del vial y ciérralo de inmediato. Introduzca la tira de prueba en el puerto de prueba con el nombre TRUEtest™ mirando hacia arriba. El medidor se activa.



**3**

Aplique la lanceta en el dedo.




**4**

Deje que se forme una gota de sangre, con la punta de la tira toque la parte superior de la gota de sangre hasta que aparezcan guiones en toda la pantalla.



**5**

En tan sólo 4 segundos, se mostrará el resultado de la glucosa.

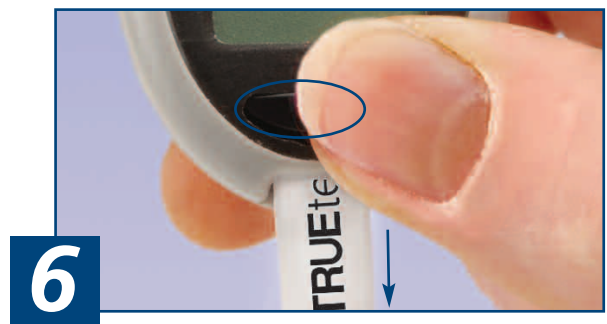

**6**

Sujete el medidor en posición vertical, con la tira de prueba mirando hacia abajo. Presione el botón de liberación para desechar la tira de prueba usada del medidor.

Esta guía proporciona una breve explicación general para realizar su prueba de glucosa en la sangre. Si necesita información más detallada sobre el procedimiento completo de las pruebas de glucosa en la sangre, consulte la sección "Pruebas de sangre". Para obtener ayuda con el uso de su Sistema para glucosa en la sangre, llame a nuestro Asistencia al Cliente al 1-800-803-6025.