
LANCING DEVICE INSTRUCTIONS FOR USE

It's As Easy as **READY, SET, LANCE!**

For obtaining capillary whole blood for testing.

Note: Lancing Device and Clear End Cap are for self-testing and intended for use by one patient only. Not suitable for use by healthcare or care workers.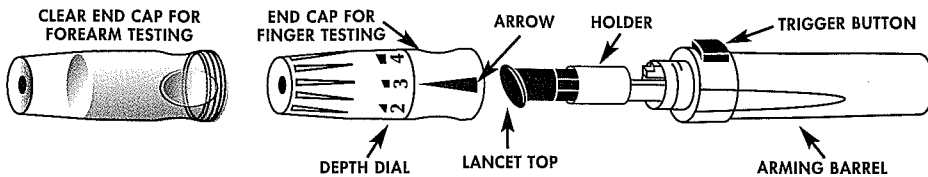

- R E A D Y -

1 Select area to be lanced. Wash with soap and warm water, rinse and dry thoroughly.

2 Twist off End Cap. Push Lancet firmly into Holder.

3 Remove Lancet Top. **Note: Save Top for safe disposal of used Lancet.**

4 Replace End Cap. **Note: Use clear End Cap for lancing forearm.**

Important Notes Regarding Forearm Samples Used in Testing Blood Glucose:

- Check with your Doctor or Diabetes Healthcare Professional to see if forearm testing is right for you.
- Blood glucose results from forearm are not always the same as results from fingertip. Do not use forearm for testing when blood sugar is rapidly rising or falling, e.g., within 2 hours of eating, after taking insulin, immediately after exercise, or when you are ill or under stress. Use fingertip sample for testing.
- Do not use forearm samples for testing if you think your blood sugar is low, if you don't regularly experience symptoms when your blood glucose is low, or if your results from forearm samples do not match the way you feel. Use fingertip sample for testing.¹


- S E T -

For Finger Testing:

- 1 Turn Depth Dial to desired lancing depth.
Note: Finer adjustments can be made by setting Arrow between numbers.

Lancing Depth Guide:

- 1-2 for soft / thin skin
- 3-4 for average skin (shown)
- 4-5 for thick / calloused skin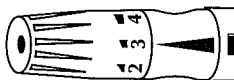


2 Lower hand to waist level. Gently massage finger to get blood flowing.

3 Hold End Cap in one hand and Arming Barrel in other. Gently pull Arming Barrel until a click is heard.

For Forearm Testing:

- 1 Use Clear End Cap in place of regular Depth Dial End Cap, but if Clear End Cap is not available, select #5 on Depth Dial.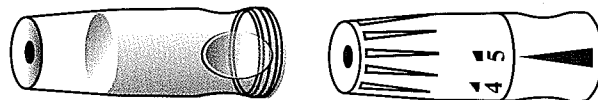


2 Rub area vigorously or apply warm compress to stimulate blood flow.

3 Hold End Cap in one hand and Arming Barrel in other. Gently pull Arming Barrel until a click is heard.

- L A N C E -

1 Place Tip firmly against finger on other hand.
Note: For best results, lance in shaded areas.

Note: For best results, lance in shaded areas.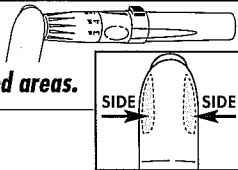


2 Press Trigger Button.

3 Gently massage finger from palm to fingertip to form blood drop.

1 Place Tip firmly against forearm.
Note: For best results, lance in shaded area.

2 Press Trigger Button.

3 Apply a firm pressure on the Lancing Device until an adequate blood drop is formed.

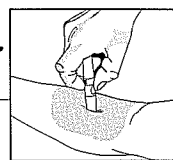• Refer to Owner's Booklet for testing instructions. •

Cleaning Lancing Device:

1. Remove End Cap.
2. Wash Cap with soap and warm water, rinse and dry thoroughly.
3. Replace End Cap.
4. Wipe outside of Lancing Device with 70% rubbing alcohol or mild soap and water.

Removing Used Lancet:

1. Twist off End Cap of Lancing Device.
2. Using saved Top, recap Lancet.
3. Remove from Holder and discard Lancet in appropriate container.

SYMBOLS LEGEND:

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Single Use Only

Attention! See Instructions for Use

LOT Lot Number

IVD For *in vitro* Diagnostic Testing Only

REF Catalog Number

For Assistance Call

1. Food and Drug Administration 2006, Diabetes Information - Glucose Meters and Diabetes Management, New Technologies: Alternative Site Testing [Electronic Version]. Retrieved February 17, 2006 from www.fda.gov/diabetes/glucose.html#11.

© 2008 Home Diagnostics, Inc. The Home Diagnostics logo is a trademark of Home Diagnostics, Inc. • Made in China

HOMEdiagnostics

Fort Lauderdale, FL 33309 USA
www.homediagnostics.com
800-803-6025 954-677-9201

U.S. and International Patents Pending. A2HD106 Rev. 4

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